IN240400075V02_GL

713-034V01

EN_IMPORTANT, RETAIN FOR FUTURE REFERENCE: READ CAREFULLY.

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WARNING

DO NOT use if poorly attached. If you are unable to securely fasten all parts, contact a dealer or trained technician to assemble. Take care during assembly not to lose your balance when testing the connections.

DO NOT use if any parts were damaged in shipping or become worn from use. Contact your dealer for replacement and repair before further use.

ALWAYS use the toilet seat when sitting during use. Sitting off-center or directly on the bucket may cause damage or personal injury.

DO NOT use on uneven ground or with legs at uneven heights. All four legs' rubber tips MUST touch the floor at all times.

ALWAYS take care to remain balanced while sitting and rising, using both armrests.

DO NOT exceed this product's weight limit: 330 pounds or 150 kg.

NOT for use by children. Keep away from children to reduce the risk of damage or injury.

MAINTENANCE

Clean & dry the bucket promptly after each use.

After shower, please wipe the chair clean and keep it dry.

Store in a dry-well ventilated place away from direct sunlight.

Place the bucket into the seat frame.

Place the toilet seat onto the frame so that the plastic clamps rest on the rear cross frame on each side of the bucket. Push down until the brackets are fully attached to the back cross frame. Check by swiveling the seat upward and then back downward.

When it is necessary to adjust the height of the legs, loosen the plastic knobs at the joints, make the necessary adjustments, and retighten the knobs.

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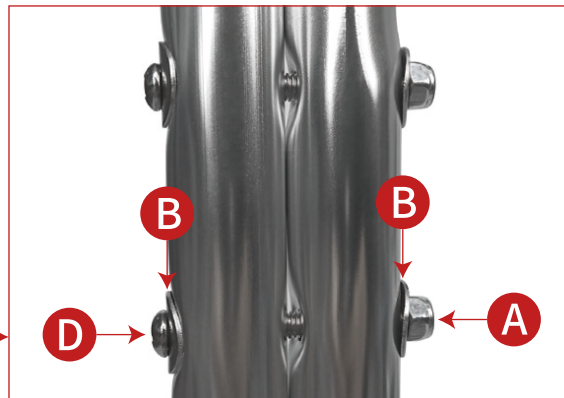
EN_Joint two ARMRESTS (1) or BACKREST WITH HANDLES (2), with 1PC SCREW (D), 2PCS METAL GASKETS (B) and 1PC SCREW NUT (A) in one hole of each side (each side has two holes).

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③ x 1

EN Put BUCKET (3) on the BUCKET HANGING BRACKET (8).

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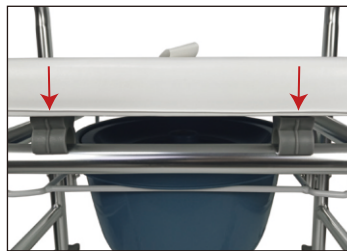
④ x 1

EN Put PADDED SEAT (4), jointing ALUMINUM FRAME (6) to two buckles.

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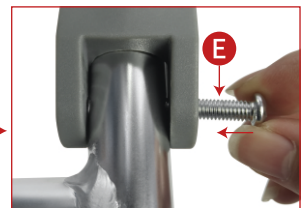
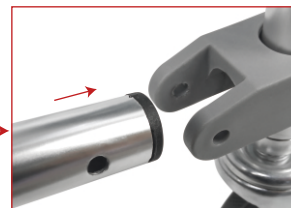
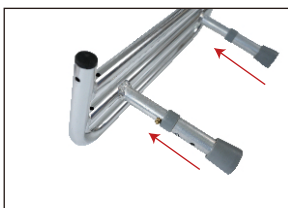
⑦ x 2

⑧ x 2

EN Put PEDAL (5) on the anchor of ARMRESTS (1), align with the hole, insert the screw and install the nut on the other side

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If you have any questions, please contact our customer care center, Our contact details are below:

US

 001-877-644-9366

 customerservice@aosom.com

Imported by Aosom LLC
27150 SW Kinsman Rd Wilsonville, OR 97070 USA
MADE IN CHINA

If you have any questions, please contact our customer care center, Our contact details are below:

CA

 416-792-6088

 customerservice@aosom.ca

Imported by Aosom Canada Inc.
7270 Woodbine Avenue, Unit 307, Markham,
Ontario Canada L3R 4B9
MADE IN CHINA

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If you have any questions, please contact our customer care center, Our contact details are below:

UK

 0044-800-240-4004

 enquiries@mhstar.co.uk

IMPORTER ADDRESS:
MH STAR UK Ltd
1 Northampton Cross Logistics Park
Northampton Cross Road
Northampton
NN4 9FH
MADE IN CHINA

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MH Handel GmbH