EN_IMPORTANT, RETAIN FOR FUTURE REFERENCE: READ CAREFULLY.

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WARNING

DO NOT use if poorly attached. If you are unable to securely fasten all parts, contact a dealer or trained technician to assemble. Take care during assembly not to lose your balance when testing the connections.

DO NOT use if any parts were damaged in shipping or become worn from use. Contact your dealer for replacement and repair before further use.

ALWAYS use the toilet seat when sitting during use. Sitting off-center or directly on the bucket may cause damage or personal injury.

DO NOT use on uneven ground or with legs at uneven heights. All four legs' rubber tips **MUST** touch the floor at all times.

ALWAYS take care to remain balanced while sitting and rising, using both armrests.

DO NOT exceed this product's weight limit.

NOT for use by children. Keep away from children to reduce the risk of damage or injury.

MAINTENANCE AND CARE

Clean & dry the bucket promptly after each use.

After shower, please wipe the chair clean and keep it dry.

Store in a dry-well ventilated place away from direct sunlight.

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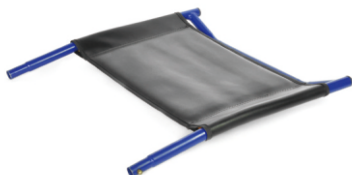
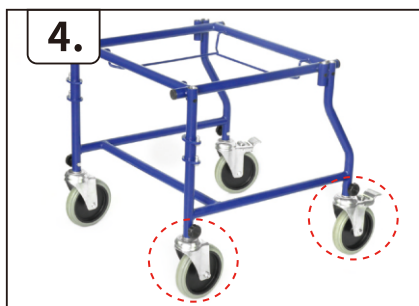
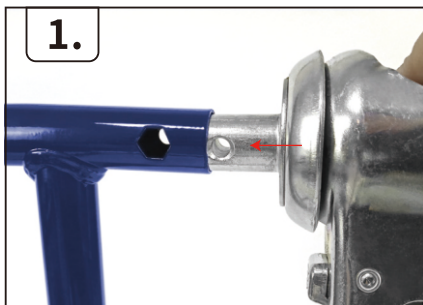
⑨ x 4

⑩ x 4

EN_Install Rear wheel and Front wheel on mainframe respectively

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EN_Note the position of the front and rear wheels

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⑥ x 2

EN_Insert armrest into mainframe and make sure the marble is stuck in the hole

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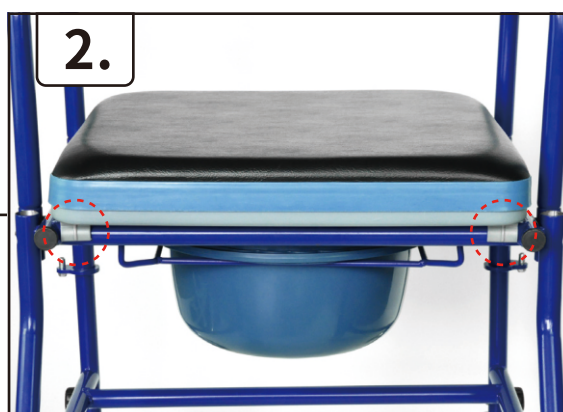
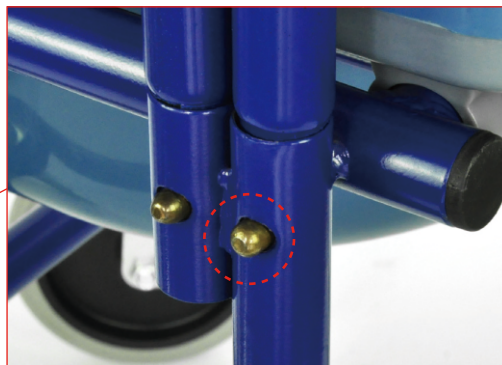
7 x 1

EN_Insert backrest into mainframe and make sure the marble is stuck in the hole

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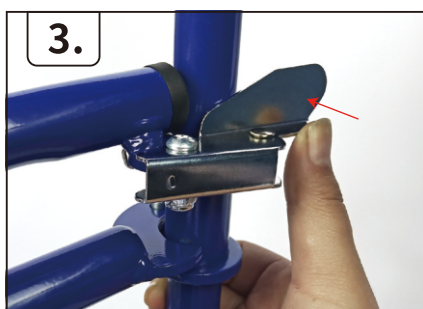
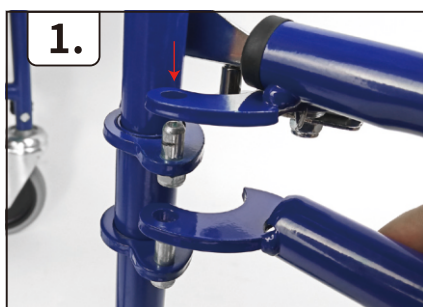
8 x 2

EN_Put footrest on the anchor of the mainframe, align with the hole slide it down and turn inward until it is tightened

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Done

If you have any questions, please contact our customer care center, Our contact details are below:

US

 001-877-644-9366

 customerservice@aosom.com

Imported by Aosom LLC
27150 SW Kinsman Rd Wilsonville, OR 97070 USA
MADE IN CHINA

If you have any questions, please contact our customer care center, Our contact details are below:

CA

 416-792-6088

 customerservice@aosom.ca

Imported by Aosom Canada Inc.
7270 Woodbine Avenue, Unit 307, Markham,
Ontario Canada L3R 4B9
MADE IN CHINA

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If you have any questions, please contact our customer care center, Our contact details are below:

UK

 0044-800-240-4004

 enquiries@mhstar.co.uk

IMPORTER ADDRESS:
MH STAR UK LTD
Unit 27, Perivale Park,
Horsenden lane South
Perivale, UB6 7RH
MADE IN CHINA

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